




## Cable Connector Cleaning Kit

### Cleaning Instructions (as needed)

1. Wipe monitor plug and cable with water or alcohol.
2. Clean outside of the connector with soap and water.
3. Apply water or alcohol to a swab (included).
4. Insert swab inside cable connector cone as shown below.
5. Rotate swab and move it around inside cone.

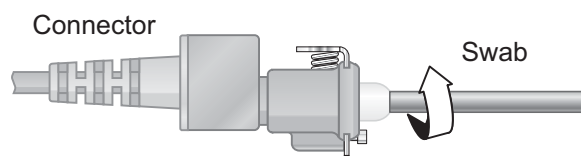

### **DO NOT SUBMERGE OR SOAK CABLE CONNECTOR!**

If swabs other than those included are used, take care to avoid disturbing the transducer protecting gel inside the connector.

REF Catalog No.  
**IPC-SWAB**

P/N 050-0096 Rev. F



CLINICAL  
INNOVATIONS

747 West 4170 South  
Murray, Utah 84123  
Phone 888-268-6222  
Fax 801-266-7373

[www.clinicalinnovations.com](http://www.clinicalinnovations.com)