

ClearView®

Uterine Manipulator

ClearView® Uterine Manipulator, with its 210° range of motion and rotating handle facilitates one-handed control, enabling the surgeon to manipulate the uterus independently.



A UTERINE MANIPULATOR



Clear Visibility

ClearView provides 120° anteversion and 90° retroversion as well as lateral movement for clear visibility.



Single-Handed Control

The self-holding rotating control handle allows the surgeon to maneuver the device independent of an assistant.



Single-Use

ClearView is completely disposable.



Patient Positioning

ClearView pivots at the cervical os and allows the patient to remain in the supine position.

PACKAGED WITH WHAT YOU NEED

ClearView is packaged in a sterile tray with everything you need to complete the procedure.

Chromotubation

The dye-injection port allows for chromotubation if needed.

Uterine Control

The included 12ml syringe inflates the tip balloon.



Disposable Sound-Dilator

The included sound-dilator helps measure and prepare the cervix.

Two Sizes

The spacer shortens the tip by 1-cm, giving you two sizes in one device.

DOING A LAPAROSCOPIC HYSTERECTOMY?

ClearView® Total

Uterine Manipulator

ClearView® TOTAL has the same features as ClearView with the addition of a pre-attached occluder balloon and three different sized colpotomy cups. The occluder balloon prevents the loss of pneumoperitoneum once the colpotomy incision is made during laparoscopic hysterectomy.



4cm



3.5cm




3cm

CLEARVIEW SET UP & USE



Check Balloon

Use supplied syringe to test the balloon prior to insertion.

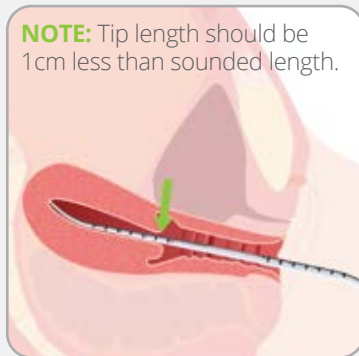

NOTE: Tip length should be 1cm less than sounded length.


Dilate & Sound Uterus

Sound the uterus. If necessary, dilate the cervix.



Insert Device & Inflate

Lubricate distal tip and insert into uterus. Slowly inflate

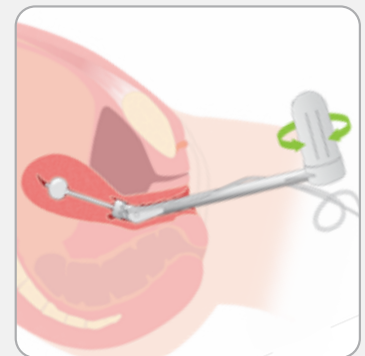

Control Handle

Rotate the handle to manipulate the uterus

See Instructions for Use for full instructions, warnings, contraindications, and precautions.

PRODUCT SUPPORT

Ordering Information

Product Description	Product Code	Box Quantity
ClearView Uterine Manipulator Kit- 7 cm tip	UM-700	5/ea
ClearView Uterine Manipulator Kit- 9 cm tip	UM-900	5/ea
ClearView TOTAL Kit- 7 cm tip	UM-750	5/ea
ClearView TOTAL Kit- 9 cm tip	UM-950	5/ea
Disposable Sound- Dilator (Sterile)	DSD-1000	50/ea

Local Distributor:

clinicalinnovations.com

6-9 The Square, Stockley Park
Heathrow | UB11 1FW | United Kingdom
Phone +44 333 370 4408 | Fax +44 (0) 845 299 1517

© Clinical Innovations, LLC. All rights reserved. | ART-0117 Rev. 01

