



Nasal & Oral Aspirators

INSTRUCTIONS FOR USE

**VPBB15011100, VPBB15012100,
VPBB15013100, VPBB15014100,
VPBB15015100, VPBB15016100,
VPBB150112100, VPBB150113100,
VPBB150114100, VPBB150115100**

Indications for Use

BoogieBaby is a non-invasive suction alternative for the nasal cavity for use with newborns, infants and children. Indicated for use as a vacuum-powered suction device to aspirate fluids from the mouth and nares.

Instructions for Use

Single Patient Use, Non-sterile.

For oral and nasal use only. Packaged clean & ready to use.

1. Connect to standard suction line.
2. Set suction pressure and use per hospital protocol.
3. Suction sterile water through device after each use.
4. Replace at least every 24 hours.
5. Dispose based on local state regulations and hospital protocol.

WARNINGS

Inserting the tip too far into the mouth or nares can cause tissue trauma. The suction device & cover should be kept free of debris. Federal Law (USA) restricts sale of this device to, or on the order of a physician.

Discontinue use of the device, if the following harms occur:

- Mucosal Hemorrhage
- Lacerations of Nasal Turbinate
- Nasal irritation/bleeding
- Hypoxia/hypoxemia
- Cardiac dysrhythmias
- Allergic Reaction

CAUTION







- Do not use tips without cleaning in between use to minimize contamination. Ensure tip is clean and free of debris in between use to prevent any unintended bacteria being introduced into the patient's nares and/or mouth leading to infection.
- Do not use if the packaging is damaged. Discard the device and obtain a new device.





Product Specifications

For use by Healthcare professionals only per the hospital protocol. Packaged in individual polyethylene bags. Distributed by Clinical Innovations LLC. Manufactured in China at an FDA registered facility.

Part Number	Description	Units/Box
VPBB15011100	BoogieBaby Nasal & Oral Aspirator - Preemie tip	100
VPBB15012100	BoogieBaby Nasal & Oral Aspirator - Preemie tip with cover	100
VPBB15013100	BoogieBaby Nasal & Oral Aspirator - Standard tip	100
VPBB15014100	BoogieBaby Nasal & Oral Aspirator - Standard tip with cover	100
VPBB15015100	BoogieBaby Nasal Aspirator - Flair tip	100
VPBB15016100	BoogieBaby Nasal Aspirator - Flair tip with cover	100
VPBB150112100	BoogieBaby Nasal Aspirator - Olive tip 4.5 inch	100
VPBB150113100	BoogieBaby Nasal Aspirator - Olive tip 3.25 inch	100
VPBB150114100	BoogieBaby Nasal Aspirator - Olive tip 4.5 inch with cover	100
VPBB150115100	BoogieBaby Nasal Aspirator - Olive tip 3.25 inch with cover	100

Symbols Glossary

Symbol	Symbol Description
	Indicates Device Manufacturer Includes name and address of the manufacturer
	Date of Manufacture
	Lot Code
	Catalog Number
	Do Not Reuse
	Consult Instructions for Use

Symbol	Symbol Description
	Do not use if the product sterile barrier system or its packaging is compromised
R_x ONLY	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician
	Warning or caution
	Not manufactured with natural rubber latex
	Non-Sterile



**CLINICAL
INNOVATIONS**

Distributed by:

Clinical Innovations, LLC
747 West 4170 South,
Murray, Utah 84123, USA
Toll Free: 1.888.268.6222
www.clinicalinnovations.com

Australia Sponsor:

Clinical Innovations Australia
36 Matheson Street, Virginia, Qld, Australia
Phone: +(07) 3862 2000



Manufactured by:

Jinhua Kangdeli Medical
Appliances Co. Ltd.
No. 70 Jiayi Road, Jianshan Industry Park
Jinhua Zhejiang, China 322312
www.kdlmedi.com

ART-0325 Rev.02